DEUTSCH

Bipolar Adapter

WICHTIG!

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Bipolar adapter

IMPORTANT!

Please read all information carefully.

These instructions for use do not replace the user manual of the electro-surgical unit used! Read the user manual of the electro-surgical unit and ask Erbe or your distributor in case of doubt!

1 Intended use

The bipolar adapter connects bipolar cables from Valleylab with Erbe electro-surgical units with international 2-Pin 22 receptacles.

2 Maximum electrical capacity

The maximum electrical capacity of this product is:

- 1000 V_p

3 Safety instructions

WARNING!

This product may be used only by trained medical staff, who have been shown how to use it according to the instructions for use.

Check the product before every occasion of use!

If damaged, do not use this product!

Do not use in the presence of combustible or explosive materials!

Never lay this product on the patient or in his/her direct vicinity.

Protect this product from any form of mechanical damage! Do not throw! Do not use force!

Erbe Elektromedizin expressly warns against modifying the product. Any modification exempts Erbe Elektromedizin from any and all liability.

4 How to use

Clean and disinfect the product before using it for the first time and before each subsequent use.

The product must be fully inserted in the corresponding socket. The limit

stop must be clearly felt. Pull the product out of the socket, keeping it straight. Transverse forces can otherwise damage the socket.

The adapter can remain in the device after use.

5 Cleaning and disinfection

This product is not suitable for mechanical cleaning and disinfection!

The adapter can remain in the device after use and be cleaned and disinfected together with it. The adapter is subject to the same reprocessing recommendations as for the device being used. Please read the User Manual for your device.

It is essential to follow the cleaning agent and disinfectant manufacturers' instructions.

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